




- Biocompatible-made of all natural Chitosan-based polymers
- Minimizes bleeding and edema after surgery
- Easily conforms to nasal anatomy
- Offers easier placement with a superior field of view
- No special storage conditions required

PRODUCT DESCRIPTION:

PosiSep® X is a hemostatic intranasal splint designed to minimize bleeding and edema and to prevent adhesions between the septum and the nasal cavity. PosiSep® X is placed in the nasal cavity after surgery or trauma. The PosiSep® line is constructed of a patient-comfortable sponge which is manufactured from naturally occurring Chitosan polymers. Subsequent irrigation will cause the product to fragment, thus optimizing patient tolerance and comfort.

- Fragments by way of gentle irrigation and aspiration
- Easy to apply-no mixing or additional prep time necessary

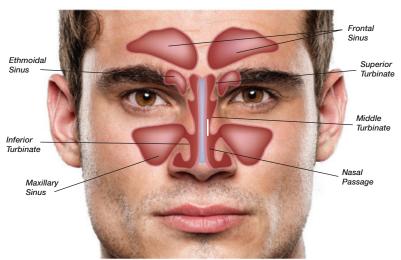

*diagram not to scale

PosiSep® X Hemostat Dressing/Intranasal Splint is indicated for use in patients undergoing nasal/sinus surgery as a space occupying splint and hemostat to:

- Separate tissue or structures compromised by surgical trauma;
- Separate and prevent adhesions between mucosal surfaces during mesothelial cell regeneration in the nasal cavity;
- Help control minimal bleeding following surgery or trauma;
- Help control minimal bleeding following surgery or nasal trauma by tamponade effect, blood absorption and platelet aggregation; and
- Act as an adjunct to aid in the natural healing process.

PosiSep is indicated for use as a nasal hemostat to treat epistaxis. PosiSep is intended for use under the direction of a licensed healthcare provider.



Before Hydration After Hydration







When PosiSep X is hydrated, it expands to fill the space preventing post-op adhesions.

What is Chitosan?

shells of crustaceans.

Chitosan (ky-toe-san) comes from a substance known as chitin, an amino-polysaccharide removed from the powdered CHITOSAN TECHNOLOGY

For more information on PosiSep® products please contact your distributor

ORDERING INFORMATION Posisep®X is available in three sizes. See ordering information below.

Part #: **Description:** 9215674 PosiSep® X Hemostat Dressing/Intranasal Splint 1.5 cm x 5.1 cm/8pk 9215664 PosiSep® X Hemostat Dressing/Intranasal Splint 1.3 cm x 3.8 cm/8pk 9215686 PosiSep® X Hemostat Dressing/Intranasal Splint 8.0 cm x 1.8 cm/8pk

How Supplied:

PosiSep® X is packaged eight (8) sterile, single use units per carton. One (1) sterile unit contains one (1) Hemostat Dressing/Intranasal Splint. PosiSep® is terminally sterilized by gamma irradiation.

