# INHEALTH

TECHNOLOGIES®

## Blom-Singer® Nasal Septal Perforation Prosthesis

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## SYMBOLS GLOSSARY (USA only)

SYMBOL	STANDARD No.	SYMBOL TITLE & REF No.	MEANING OF SYMBOL
	EN ISO 15223-1:2016	Manufacturer; 5.1.1	Indicates the medical device manufacturer, as defined in EU Directives or Regulations
	93/42/EEC EU 2017/745	CE Marking; 93/42/EEC Annex XII, EU 2017/745 Annex V	CE marking of conformity
	EN ISO 15223-1:2016	Authorized representative in the European Community; 5.1.2	Indicates the authorized representative in the European Community
	EN ISO 15223-1:2016	Catalogue number; 5.1.6	Indicates the manufacturer's catalogue number so that the medical device can be identified
	EN ISO 15223-1:2016	Batch code; 5.1.5	Indicates the manufacturer's batch code so that the batch or lot can be identified
	EN ISO 15223-1:2016	Use-by date; 5.1.4	Indicates the date after which the medical device is not to be used
	EN ISO 15223-1:2016	Caution; 5.4.4	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself
	EN ISO 15223-1:2016	Do not use if package is damaged; 5.2.8	Indicates a medical device that should not be used if the packaging has been damaged or opened
	EN ISO 15223-1:2016	Consult instructions for use; 5.4.3	Indicates the need for the user to consult the instructions for use
	ISO 7000:2019	Packaging unit; 2794	To indicate the number of pieces in the package

**SYMBOLS GLOSSARY** (USA only)

<b>SYMBOL</b>	<b>STANDARD No.</b>	<b>SYMBOL TITLE &amp; REF No.</b>	<b>MEANING OF SYMBOL</b>
	EN ISO 15223-1:2016	Upper limit of temperature; 5.3.6	Indicates the upper limit of temperature to which the medical device can be safely exposed.
	ISO 7000:2019	Single patient multiple use; 3706	To indicate that the medical device may be used multiple times (multiple procedures) on a single patient

# INHEALTH

TECHNOLOGIES®

---

## UNITED STATES

1110 Mark Avenue  
Carpinteria, CA 93013-2918 USA  
Toll-Free: 800.477.5969  
Toll-Free Fax: 888.371.1530

## INTERNATIONAL

Tel: +1.805.684.9337  
Fax: +1.805.684.8594  
E-mail: [order@inhealth.com](mailto:order@inhealth.com)

## ORDER ONLINE

[www.inhealth.com](http://www.inhealth.com)

### Freudenberg Medical, LLC

1110 Mark Avenue  
Carpinteria, CA 93013-2918 USA  
[www.inhealth.com](http://www.inhealth.com)

### EMERGO EUROPE

Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

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Blom-Singer® and InHealth Technologies® are registered trademarks of Freudenberg Medical, LLC.
Copyright © 2020 Freudenberg Medical, LLC. All rights reserved.